

# CAST™ WALKER

## Application Instructions:

Universal design fits most casts and can be used on either foot. Can be trimmed to fit a smaller foot.

Applies quickly and easily with a special adhesive which sets in just 45 seconds. Just follow these simple instructions:

1. Must wear impermeable rubber gloves! (DO NOT use cotton gloves.)
2. An additional fiberglass splint within the bottom of the cast is recommended.
3. Mold CastWalker to the bottom of the cast while the fiberglass hardens for 30 seconds. Alignment guidelines include:
  - The high wall on the concave side is medial.
  - Place the CastWalker as far forward as the patient's arch will allow.
  - Trim off any excess CastWalker material protruding distal to the cast with regular scissors.
4. Apply the entire tube of adhesive to the concave side of the CastWalker.
  - Spread the adhesive over the entire surface of the concave side of the CastWalker.
  - Press CastWalker firmly on the prepared bottom of the cast. The highest side must be medial and slightly distal to the highest point of the longitudinal arch of the foot.
5. Hold for 45 seconds to set the adhesive. Allow the patient to ambulate after 5 minutes.



For Pediatric use or smaller casts, CastWalker can easily be trimmed with scissors.



CastWalker Adhesive (included) assures a secure bond to the bottom of the cast.

## Purchasing Information:

CastWalker is sold in individual kits (three kit minimum) or in a case of ten kits. Specify pediatric, regular or extra large.

Each CastWalker includes one CastWalker, one tube of special CastWalker adhesive, and complete application instructions.

For the name of your nearest CastWalker dealer, call a customer service representative at **1-800-254-5438** or **1-847-328-9541**

Patent No. 5,433,695